# BAUSCH+LOMB

## EasySept® Patient Insert

Please read carefully and keep this package insert for future reference. EasySept Peroxide Solution disinfects, removes protein & stores all soft contact lenses, including silicone hydrogels

### WARNINGS:

-  Only use the enclosed special lens case and neutralising disc with EasySept Peroxide Solution. **DO NOT USE WITH OTHER LENS CASES.**
-  NEVER insert EasySept Peroxide Solution directly into the eye. **DO NOT place your lens into the eye until complete neutralisation of the solution has occurred (minimum 6 hours).**
-  Do not rinse lenses with EasySept Peroxide Solution prior to inserting lenses into your eyes. If you want to rinse your lenses, use a sterile saline solution.

### DIRECTIONS FOR USE

-  Always wash and dry your hands before handling your lenses and after using EasySept.
-  Check that the special lens case includes the neutralising disc.
-  Place your left and right lenses on the domes inside the appropriate baskets. Close the baskets taking care not to trap the lenses.
-  Hold the lens case lid and rinse the lenses in the baskets with EasySept Peroxide Solution for 5 seconds each, over a sink/basin.
-  Fill the lens case up to the fill line with fresh EasySept Peroxide Solution then immediately place the holder basket into the lens case. Do not invert or shake case.
-  Your lenses will be ready to wear after soaking for 6 hours.
-  Once lenses are removed from the baskets, discard any remaining solution from the lens case, rinse the case with EasySept Peroxide Solution over a sink/basin, and leave the lid off the case to air dry.

If you do not intend to wear your lenses immediately after the disinfection/neutralisation process, repeat the steps above after 24 hours and before inserting your lenses. If a separate daily cleaner is used, be sure to rinse the lenses thoroughly with saline before soaking in EasySept® Peroxide Solution.

### CONTENTS

Sterile and buffered 3% hydrogen peroxide solution with a phosphonic acid stabilizer, sodium chloride and phosphate buffers.

### Contraindications (Reasons not to use):

If you are allergic to any ingredient in this product, do not use. **Failure to discard solution from lens case after each use or use of water to care for your lenses may lead to contamination resulting in eye injury and potential loss of vision. See accompanying instructions for additional important safety information.**

### Important Safety Information:

- Always follow the product directions for use. Failure to follow product directions may lead to vision loss or will result in burning and stinging.
- EasySept Peroxide Solution **IS NOT** a multi-purpose solution.
- Do not insert EasySept Peroxide Solution or the lens directly into the eye until complete neutralisation of the solution has occurred (minimum 6 hours).
- Do not rinse lenses with EasySept Peroxide Solution prior to inserting lenses into your eyes. If you want to rinse lenses, use a sterile saline solution.
- Always wash and dry hands before handling lenses.
- Do not use tap water, bottled water or saliva with lenses or lens case.
- Only use fresh solution to clean and disinfect contact lenses.
- Always replace your solutions, lenses and lens case as directed.
- To avoid contamination, do not touch tip of container to any surface. Replace cap after using.
- Not for use with heat (thermal) disinfection.
- Visit your eye care practitioner regularly.

### Precautions:

- Always discard solution from lens case after each use.
- Keep the bottle tightly closed when not in use.
- Use before expiration date printed on the carton and bottle.
- Discard bottle and lens case 90 days after opening or after 35 uses, whichever occurs first.
- Store at room temperature.
- Do not use if tamper evident seal is missing or broken.
- Only use solution with specially designed EasySept lens case.
- Do not invert or shake case as residual unneutralised peroxide could remain on the cap.
- Avoid contact of EasySept Peroxide Solution with fabrics.

### KEEP OUT OF REACH OF CHILDREN

How supplied: EasySept Peroxide Solution is available for purchase in sterile, 120 and 360 mL plastic bottles and supplied with a new peroxide lens case. Bottles and cartons are marked with a lot number and expiration date.

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©Bausch & Lomb Incorporated 151083975-02
®/TM are trademarks of Bausch & Lomb Incorporated or its affiliates.